

Dimensions & Weight

Overall length	228 mm
Overall diameter	Ø88 mm
Dry weight system	240 grams

Connection

To mains	1/2" BSP
----------	----------

Storage & Handling

- Keep dry during storage
- Protect against freezing after first use
- Handle with care; do not expose to shocks
- Disposable product; discard as regular waste
- Refer to manual for detailed instructions

Product name Productcode

H ₂ OK Medical ShowerFilter	31.2.101
Replacement Cartridge	31.2.151

Packaging

Sterilized by gamma irradiation	
Starterset	12 pcs./box
Replacement cartridge	12 pcs./box



Filtrix BV

Siliciumweg 28 • 3812 SX Amersfoort • Nederland

T +31 33 422 04 40 • F +31 33 422 04 41

E info@filtrix.com • I www.filtrix.com

Please visit our website to obtain information about your local support!




Technology

Capillary microfiltration membranes

Performance¹

Lifetime	35 days maximum
Flowrate	see graph below, 6 l/min flow restrictor recommended for water saving purposes

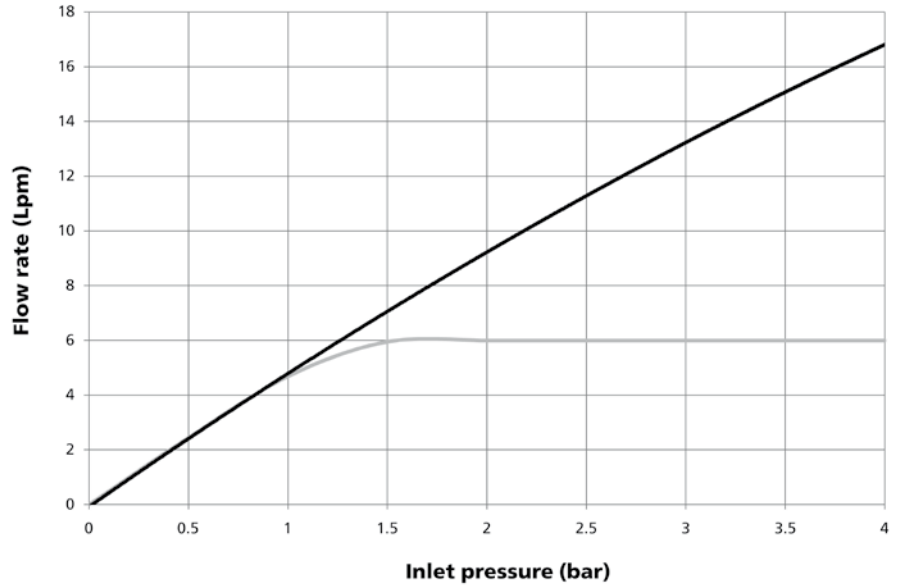

— Without flow restrictor
— With flow restrictor

Compliance

Biological retention ²	bacteria > log 6 fungi > log 4
Mechanical effects	turbidity reduction
Material safety	components approved according to NSF/KTW
Structural integrity ³	hydrostatic pressure 8 bar

Operating conditions

Max. operating pressure	5 bar
Operating temperature	min. 0 °C; max. 60 °C 70 °C for 30 min. during lifetime
Replacement interval	35 days (disposable product)
Complies with standard thermal and chemical disinfection procedures	

¹ Product performance is dependent on incoming water conditions

² Tested according to ASTM F838-05 using *Pseudomonas dimunita* by Vitens Laboratory, The Netherlands

Tested based on NSF P231 using *Klebsiella terrigena* by Vitens Laboratory, The Netherlands

Tested based on NSF P231 using *Legionella pneumophila* by Vitens Laboratory, The Netherlands

Tested based on NSF P231 using *Aspergillus fumigatus* by Vitens Laboratory, The Netherlands

³ Tested according to NSF 42

Disclaimer: The information and data in this document is for general purposes only. The information is believed to be correct, however any reliance you place on such information is strictly at your own risk. While we endeavor to keep the information accurate, we make no representations or warranties of any kind, express or implied, about the information, products, services, or related graphics on this Data Sheet for any purpose. Since the conditions under which our products may be used are beyond our control, we cannot accept any liability with respect to the use of our products. Warranty information for the products can be found in our Terms and Conditions of sale. Property rights (industrial, intellectual, and other) must be observed.

Norit H₂OK Medical ShowerFilter